
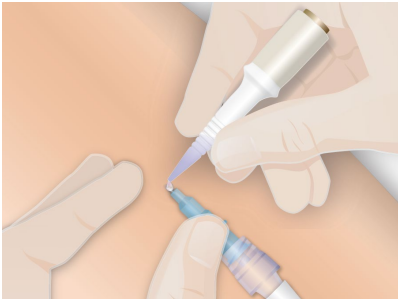

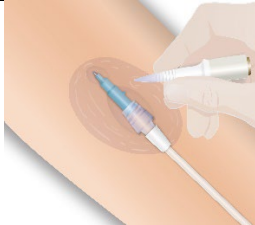
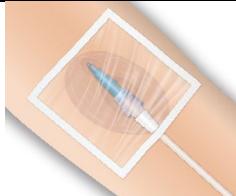


## SecurePortIV® Adhesive In-Service Guide Peripheral IV Application

### Product Preparation

1. Peel open pouch and remove applicator or drop it onto sterile field.
2. Prep skin and insert catheter per facility protocol.
3. Prior to application of SecurePortIV® adhesive, ensure all skin prep solutions are dry and remove any moisture or blood at insertion site with sterile gauze.

<p>Point the tip towards the ceiling and away from the patient. Press the bottom of the applicator upward.</p>	
<p>Invert the applicator and gently squeeze the ridges to initiate adhesive flow. Place 1-2 drops of adhesive at the catheter-skin junction.</p> <p><b><i>Allow each drop of glue to “set-up” for a few seconds before adding the next drop.</i></b></p> <p><b><i>Make sure the tissue adhesive is surrounding the entire circumference of the catheter.</i></b></p>	
<p>Place an additional 1-2 drops of adhesive under the catheter hub/extension set connection.</p> <p>Apply gentle pressure for 30 seconds to ensure securement between the catheter and the skin surface.</p>	
<p><b><i>As desired, the remaining adhesive can be spread out beyond the area of the catheter to increase dressing adherence.</i></b></p>	
<p>After application of SecurePortIV® adhesive, a transparent film dressing should be applied per facility protocol.</p>	

### Removal

Removal of adhesive occurs naturally over 5 – 7 days. Earlier removal may be accomplished by gently pressing any hydrocarbon based adhesive remover pad over the insertion site for 30 seconds to one minute to saturate the area where adhesive is present. This will soften the adhesive and allow easy removal of the device. To avoid skin irritation do not attempt to rub the area with the adhesive remover.